

Canine Parvovirus, Canine Coronavirus, and Giardia Ag Combined Test Kit

Catalogue No: RAPG-CPVCCVGAT-001

TEST KIT DESCRIPTION

The Biopanda Canine Parvovirus, Canine Coronavirus, and Giardia Ag Test kit qualitatively detects Canine Parvovirus (CPV Ag), Canine Coronavirus (CCV Ag), and Giardia antigen from a dog's faeces or vomit. This test is a sandwich lateral flow immunochromatographic assay and results can be obtained between five and ten minutes.


The cassette has three test windows with invisible Test(T) and Control(C) regions that become visible during sample testing. If enough CPV/CCV/Giardia antigen is present in the sample, the T band will become visible in the corresponding test window. The C band will appear during all tests to clarify a valid result.

KIT CONTENTS

- 10 foil wrapped pouches containing a test cassette, pipette and desiccant
- 10 assay buffer tubes (1.0ml)
- 10 swabs
- 1 product insert

METHOD

- Collect a sample of the dog's **faeces or vomit** using the swab.
- Place the swab sample into the assay buffer tube and stir thoroughly.
- Position the cassette horizontally and using the pipette, slowly **drip 3 drops of the sample into each well.**
- Results are read between **5-10 minutes. After 10 minutes results are considered invalid.**

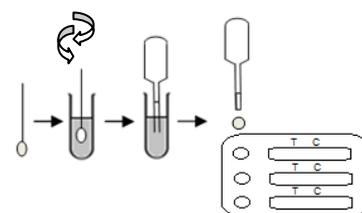


RESULTS

Positive results: Both the T and C bands are visible regardless if the T band is faint.

Negative results: Only the C band is visible.

Invalid results: No coloured band becomes visible in the C zone regardless if the T band appears.

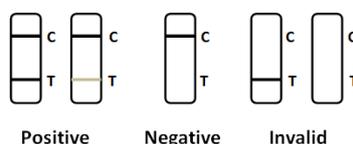

STORAGE INSTRUCTIONS

Store the kit between 2-30°C and ensure the kits are not frozen or stored in direct sunlight. The test is valid for 18 months until the expiration date printed on the foil wrapping.

SAFETY MEASURES

Follow these instructions for the best results:

- Ensure the test kit is at room temperature before running the test.
- Keep the cassette inside the foil wrapper until it is needed.
- Ensure each test is used only once.
- Test kits that have reached their expiry date should not be used.
- Only use reagents from this kit when performing the test to ensure quality controlled testing.

RESTRICTIONS

The Biopanda Canine Parvovirus Ag/Canine Coronavirus Ag/Giardia Ag Test kit is for *in vitro* veterinary diagnosis only. Consult all other clinical information and the use of other diagnostic methods, such as PCR, is recommended to ensure an accurate diagnosis.

